

URINALYSIS

(ISS MED/3A - ALL/FIN)

Page 1 of 2 pages

I

NOTE

Perform if suspected bladder infection (refer to {BLADDER/KIDNEY INFECTION} (SODF: ISS MED: A THROUGH B) as directed by Surgeon, and during all PHS exams (SODF: ISS MED: PHS).

- AMP
(blue)
1. Unstow:
 - Gloves, Non-Sterile (P3-B2)
 - Cotton Swabs (P2-B3)
 - Urine Chemstrips with Color Chart (P4-A3)
 2. To perform urinalysis using Urilux, refer to BIOCHEMICAL URINE TEST: MO-9 (RODF: SERVICE MODULE MEDICAL OPERATIONS BOOK 2), and record results in appropriate table at end of procedure.
To perform urinalysis using Urine Chemstrip, go to step 3.
 3. Don Non-Sterile Gloves.
Hold two Cotton Swabs together under urine stream to saturate.
Pass two Cotton Swabs over Urine Chemstrip to saturate colored reagent pads.
Compare with Colorchart after recommended time.
Record results in appropriate table at end of procedure.
 4. Dispose of Cotton Swabs and Urine Chemstrip in biohazardous trash.
 5. Contact Surgeon to report abnormal findings and discuss treatment.

URINALYSIS

(ISS MED/3A - ALL/FIN)

Page 2 of 2 pages

Urinalysis (CHEMSTRIP 10SG)

SpG						
pH						
Leuk						
Nit						
Protein (mg/dL)						
Glucose (mg/dL)						
Ketones						
Urobilinogen (mg/dL)						
Bilirubin						
Blood						
Hemoglobin (Ery/uL)						

OR

Urinalysis (URILUX)

SG						
pH						
LEU (Leu/uL)						
NIT						
PRO (g/L)						
GLU (mmol/L)						
KET(mmol/L)						
UBG (umol/L)						
BIL (umol/L)						
ERY (Ery/uL)						